

SINA Comfort



beka DIFFERENT
BY DESIGN



Watch product video!



Safely and comfortably shower residents and patients with high mobility dependency.

The SINA Comfort shower trolley is the ideal care aid to shower highly dependent residents and patients. The transfer from the bed to the shower trolley is done by moving sideways to the bed and moving the patient to the shower trolley using a transfer aid. The trolley padding offers a comfortable surface on which the resident rests safely. The high side rails offer optimal protection against rolling out of the shower trolley.

Due to their need for help, patients can no longer be provided with conventional transfer aids such as bath and shower lifts and then have to be washed in bed. This often does not allow for adequate, dignified personal hygiene.

Group 4 Group 5



Fold-down side rails for easy transfer



Flexible water drain hose



Adjustable rail with safety hinges
to extend the lying surface



Positive and negative steplessly adjustment of the lying surface,
applicable for a shock position



Pleasantly padded lying surface for comfortable showering,
optional head wedge for even better comfort



SINA XXL - extremely stable design
for bariatric patients up to 250 kg



Made of high quality stainless steel

Powder-coated stainless steel construction.



Negative and positive steplessly adjustable

The positive and negative tilt adjustment of the SINA Comfort enables the caregiver to adjust the lying position of the patient flexibly according to their needs. For example, the patient or resident can easily be brought to the Trendelenburg position.



Electrical or hydraulic height adjustment

Thanks to the powerful battery, the SINA Comfort is steplessly adjustable in height. Alternatively, the shower bed can also be purchased as a hydraulic version, where the height is adjusted using the foot pedal.



XXL model (up to 250 kg) available

The extra-wide lying surface and height-adjusted side rails provide the necessary basis to enable bariatric residents to shower safely and comfortably. The individually stored, extra large castors enable easy maneuvering despite the increased weight.

Options

Tiltable head part

Adjust the head board of the SINA Comfort, to ensure a comfortable lying position of your patient.



Wedge pillow

The soft wedge pillow can be used to bring the head of your patient into a higher position. It provides better adaptability and making sure your patients lie comfortably.



Technical Data

	SINA Comfort	SINA Comfort XXL
Length (optional)	1.740 mm or 2.040 mm	2.040 mm
Width (optional)	760 mm	990 mm
SWL	165 kg	250 kg
Height Adjustment	540 - 865 mm	720 - 1.020 mm
Power Supply*	24 V battery	24 V battery

*for models with electrical height adjustment

Product Information

Stepless, vertical height adjustment, either electrical or hydraulic
Positive and negative adjustment of the lying surface, suitable for shock position
Stable scissor lift system
Pleasantly padded lying surface, easy to disinfect
Powder coated stainless steel
Foldable side rail with maintenance-free safety hinges

Available Options



24 V replacement battery



Wall charger



Wedge pillow



Tiltable head part*

*only applies to SINA Comfort 200er models (length = 2.040 mm)



BEKA Hospitec GmbH | Am Rübenmorgen 3 | 35582 Wetzlar-Dutenhofen
Tel +49 641 922 20-0 | E-Mail info@beka-hospitec.com
www.beka-hospitec.com

